

Genie[®]

VPS TRAY ADHESIVE

DIRECTIONS FOR USE

For dental use only.

Caution: U.S. federal law restricts this device to sale by or on the order of a dentist.

1. PRODUCT DESCRIPTION

Genie[®] VPS Tray Adhesive is used to enhance the adhesion of vinyl polysiloxane impression materials to rigid impression trays. Use of the adhesive helps assure that the completed impression remains firmly attached to the tray upon removal from the mouth.

1.1 Delivery forms

Genie[®] VPS Tray Adhesive is available in a 14 ml bottle with integrated applicator brush in bottle cap

1.2 Composition

Pressure Sensitive Silicone Adhesive; Ethyl Acetate; Violet Colorant

1.3 Indication

Genie[®] VPS Tray Adhesive is indicated for application to impression trays to enhance the adhesion of all types of vinyl polysiloxane impression materials to intraoral impression trays including perforated and non-perforated stock metal and plastic, custom acrylic and mesh-type double arch trays.

1.4 Contraindications

None Known

2. GENERAL SAFETY NOTES

2.1 Warnings

1. Genie[®] VPS Tray Adhesive contains ethyl acetate. Do not breathe vapors. Use only in well ventilated areas. May be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons. Do not take internally.
2. Inhalation: Move subject to fresh air.
3. Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
4. Avoid skin contact to prevent irritation and possible allergic response. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention.
5. Avoid contact with oral soft tissues/mucosa to prevent inflammation. If accidental contact occurs, flush mucosa with plenty of water and expectorate water. If sensitization of mucosa persists, seek medical attention immediately.
6. Ingestion: If accidental swallowing occurs, drink lots of water. If nausea or illness develops, seek medical attention immediately.
7. **Danger:** Extremely flammable. Genie[®] VPS Tray Adhesive contains ethyl acetate. Use only in well ventilated areas. Keep away from heat, sparks and open flame.

2.2 Precautions

1. Genie® VPS Tray Adhesive is extremely flammable and a hazardous material. Dispose of in accordance with federal, state and local regulations. Wear suitable protective clothing, eyewear and gloves. Use only in well ventilated areas. Keep away from heat, sparks and open flame. (See Warnings).
2. Interactions: Do not use with a polyether, polysulfide or condensation silicone material.

2.3 Storage

Genie® VPS Tray Adhesive should be stored with original cap only. Keep out of direct sunlight and store in a well ventilated place at temperatures between 18-27°C/65-80°F. Protect from moisture. Do not freeze. Do not use after expiration date.

2.4 Adverse Reactions

1. Headache, dizziness, nausea, incoordination and respiratory irritation may occur with excessive inhalation of Genie® VPS Tray Adhesive. Eye redness, tearing and blurred vision may result from direct exposure. Vomiting, nausea, diarrhea, and irritation of the mouth, throat and stomach may result if ingested. Skin irritation may result from direct exposure and skin defatting from prolonged exposure. (See Warnings and Precautions).
2. Corneal damage may result with prolonged eye exposure to Genie® VPS Tray Adhesive. (See Warnings).
3. Allergic contact dermatitis and other allergic reactions may occur in susceptible individuals. (See Warnings).

3. STEP-BY-STEP INSTRUCTIONS

1. Prior to application of Genie® VPS Tray Adhesive, trial fit tray for proper fit and extension.
2. After proper fit is assured, rinse and thoroughly dry tray.
3. Brush a thin layer of Genie® VPS Tray Adhesive onto the tray surfaces and allow to air dry for at least five minutes, but no more than 30 minutes.
4. Mix and place impression material directly into coated impression tray.

4. HYGIENE

4.1 Cleaning and Disinfection

To prevent contamination of the bottle and cap, the use of a protective barrier is recommended.

5. LOT NUMBER AND EXPIRATION DATE

1. Do not use after expiration date. ISO standard uses: "YYYY/MM."
2. The following numbers should be quoted in all correspondences:
 - Reorder number
 - Lot number on the bottle
 - Expiration date

Manufactured for:

 **Sultan**Healthcare

Sultan Healthcare
1301 Smile Way • York, PA 17404-0807 • USA
Toll Free: 800-637-8582 • Phone: 201-871-1232
Fax: 201-871-0321 • www.sultanhealthcare.com

0077100DF, R2-080114