ENGLISH

Topex® Prophylaxis Paste – Non-Fluoride

Unit dose cup: AD60002, AD60011
Jar: AD30040

INDICATIONS FOR USE

For cleaning and polishing procedures as part of a professionally administered prophylaxis treatment.

CONTRAINDICATIONS

This product should not be used in individuals with a known sensitivity to any component of this product.

WARNINGS

Do not intentionally misuse by swallowing. For professional dental use only.

PRECAUTIONS

Use only as directed. Not for unsupervised home use. Keep out of reach of children. Do not re-use individual prophylaxis paste cups or disposable prophylaxis angles on multiple patients. If using this product from a jar, dispense a small amount necessary for the procedure and re-seal the jar. Do not dip the prophylaxis cup directly into the jar.

ADVERSE REACTIONS

No known adverse reactions

DOSEAGE AND ADMINISTRATION

BULK JARS: Dispense a small amount necessary for the procedure and re-seal the jar. Do not dip the prophylaxis cup directly into the jar.

UNIT DOSE: Each unit dose cup contains enough prophylaxis paste for (1) prophylaxis procedure. Do not re-use individual prophylaxis paste cups or disposable prophylaxis angles on multiple patients.

DIRECTIONS: Use any commercially available prophylaxis angle, rubber cup combination. To fill the cup, press the cup face into the paste to load. Apply the cup to the tooth surface and use gentle strokes with the handpiece running. Note that speeds above 3000 RPM should be avoided and use caution to avoid overheating the tooth.

Rinse excess material from the teeth and suction any remaining paste or saliva – do not allow the patient to swallow.

COMPOSITION

Medicinal ingredients: None

Non-medicinal ingredients: Glycerin, Pumice, Glycol 1450, Glycol 400, Purified water, Monosodium Phosphate, Veegum D, Agar, Sodium Benzoate, Zinc Stearate

MINI: Sodium saccharin, Mint flavor, FD&C Blue #1, FD&C Yellow #10, Titanium dioxide

CHERRY: Sucralose, Cherry flavor, FD&C Red #40, FD&C Red #22, FD&C Blue #1, Titanium dioxide

STORAGE

Store at normal room temperature 20°-25°C (68°-77°F).

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EC REP MDSS GmbH • Schiffgraben 41 • 30175 Hannover, Germany