Topex® Prophylaxis Paste – Non-Fluoride

Unit dose cup: AD60002, AD60011
Jar: AD30040

INDICATIONS FOR USE

For cleaning and polishing procedures as part of a professionally administered prophylaxis treatment.

CONTRAINDICATIONS

This product should not be used in individuals with a known sensitivity to any component of this product.

WARNINGS

Do not intentionally misuse by swallowing. For professional dental use only.

PRECAUTIONS

Use only as directed. Not for unsupervised home use. Keep out of reach of children. Do not re–use individual prophy paste cups or disposable prophly angles on multiple patients. If using this product from a jar, dispense a small amount necessary for the procedure and re–seal the jar. Do not dip the prophy cup directly into the jar.

ADVERSE REACTIONS

No known adverse reactions

DOSAGE AND ADMINISTRATION

BULK JARS: Dispense a small amount necessary for the procedure and re–seal the jar. Do not dip the prophy cup directly into the jar.

UNIT DOSE: Each unit dose cup contains enough prophy paste for (1) prophylaxis procedure. Do not re–use individual prophy paste cups or disposable prophy angles on multiple patients.

DIRECTIONS: Use any commercially available prophy angle, rubber cup combination. To fill the cup, press the cup face into the paste to load. Apply the cup to the tooth surface and use gentle strokes with the handpiece running. Note that speeds over 3000 RPM should be avoided and use caution to avoid overheating the tooth.

Rinse excess material from the teeth and suction any remaining paste or saliva – do not allow the patient to swallow.

COMPOSITION

Medicinal ingredients: None

Non-medicinal ingredients: Glycerin, Pumice, Glycol 1450, Glycol 400, Purified water, Monosodium Phosphate, Veegum D, Agar, Sodium Benzoate, Zinc Stearate
MINT: Sodium saccharin, Mint flavor, FD&C Blue #1, FD&C Yellow #10, Titanium dioxide

CHERRY: Sucralose, Cherry flavor, FD&C Red #40, FD&C Red #22, FD&C Blue #1, Titanium dioxide

PROPHY PASTE HOLDERS CLEANING AND STERILIZATION:

The Prophy Paste Holders can be cleaned and then steam sterilized, see instructions below. Discoloration may occur after repeated sterilization cycles – discoloration does not affect function.

Cleaning:

- Remove and discard the single–use prophy paste cup.
 - In an ultrasonic cleaner, prepare an instrument cleaning solution per the manu–facturer’s recommendation.
 - Immerse the holder in the ultrasonic cleaner containing the prepared solution for the contact time specified by the solution manufacturer.
 - Operate the ultrasonic cleaner in accordance with the manufacturer’s instructions.
 - Rinse the holder under running tap water for a minimum of 30 seconds and allow to dry completely.
 - Visually inspect the holder for visible soil and repeat the cleaning procedure if necessary.
 - Dry with a clean cloth or allow the holder to air dry. Proceed to Sterilization.
- Sterilization:
- Place the holder in a steam sterilization pouch. Sterilize metal and plastic compo–nents in separate pouches to avoid damage.
 - Place bagged holder into the steam sterilizer per manufacturer’s instructions.
 - Gravity Steam Sterilization: Cycle at 132C (270F) for 10 minutes
 - Pre–vacuum Steam Sterilization: Cycle at 132C (270F) for 3 minutes
 - To dry, use the drying cycle of the sterilizer. Set cycle for 20 – 30 minutes.
 - To maintain sterility, holders should remain in the sealed pouch until ready for use and stored in a dry, clean location.

STORAGE

Store at normal room temperature 20°–25°C (68°–77°F).

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