ENGLISH

### Technique for use of 3-Way® Trays

- 1) Affix I.D. Label to tray handle and write the patient name and date on the tag.
- 2) Place unloaded 3-Way® Tray into patient's mouth, making sure the crossbar and walls of the tray are clear of dentition. In case of insufficient space between the teeth and walls of the tray, a sideless tray should be substituted.
- 3) Have the patient close into centric occlusion. The patient should be informed to remember this position and be able to readily reproduce it.
- 4) Dispense heavy or regular body impression material and load both sides of the 3-Way® Tray.
- 4A) Various techniques can be employed including removal of tray with set material then relieving critical areas and relining with wash impression material.
- 5) Repeat step 2 with loaded tray.
- 6) When set, remove and inspect final impression.

### Characteristics of the 3-Way® Tray

3-Way® Trays provide the means for superior impressions while simultaneously capturing master impression, counter impression, and a bite registration all in one user-friendly step. This allows an accurate impression of both arches and a precisely articulated model. Save time by eliminating remakes...Save MONEY by spending less patient chair time.

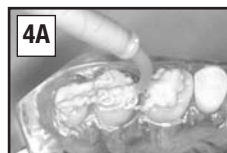
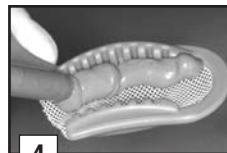
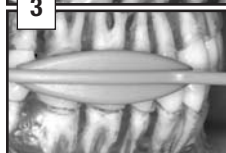
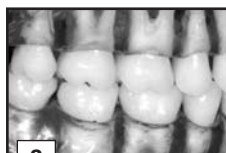
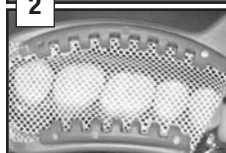
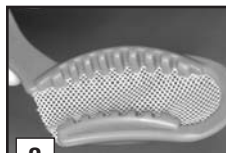
### Helpful Hints for pouring up 3-Way® Trays

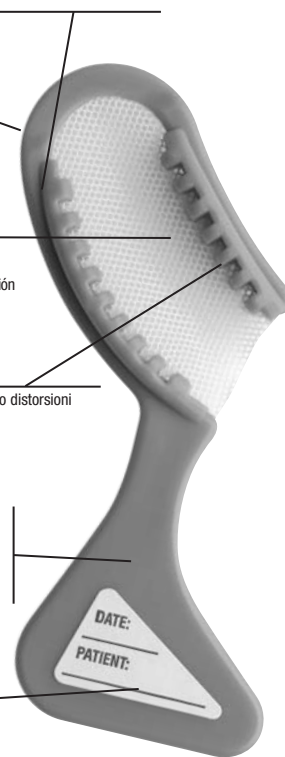
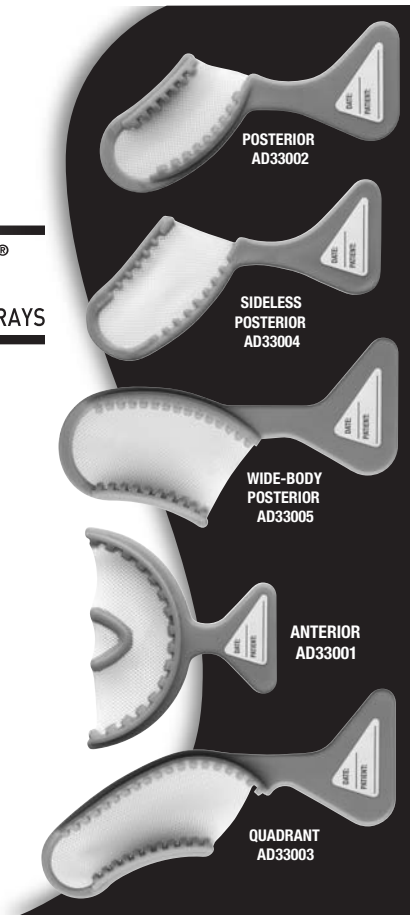
- Evaluate impression for any visible signs of tray walls showing through impression material; this may affect accuracy of impression.
- Differentiate maxillary and mandibular arches by use of I.D. label.
- Position dowel pins for dies and box impression.
- Pour working side prior to pouring counter.
- Add stone or epoxy into the impression in small increments.

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