

Buckley's Formocresol

DIRECTIONS FOR USE

REF – 10203 – Buckley's Formocresol – 1oz/30mL

NPN number: Buckley's Formocresol - 00470155

For dental use only.

1. DESCRIPTION

Buckley's Formocresol: 19% formaldehyde, 35% cresol, 17.5% glycerine.

2. INDICATIONS

Indicated as a disinfectant and devitalizing solution used as part of endodontic treatment.

3. CONTRAINDICATIONS

This product should not be used with individuals with known sensitivities to formaldehyde or cresol. This product should not be used with individuals who are pregnant or suspected to be pregnant without a prior physician consultation.

4. PRECAUTIONS

This product can cause severe burns of skin and mucosa. Take precautions protect skin, mucosa and eyes during use.

5. WARNINGS

Do not swallow. Do not allow material to come into contact with intact skin or mucosa. Wear eye protection when handling. Ensure proper tooth isolation, including rubber dam when applying. Keep out of reach of children. For professional dental use only.

6. DOSAGE AND ADMINISTRATION

1. Intracanal disinfection in permanent teeth: Isolate the tooth with a rubber dam. Clean, de-bridge and instrument the canals. Rinse and dry the canal spaces. Moisten a small cotton pellet with Buckley's Formocresol. Blot or wring any excess liquid from the pellet. Place the cotton pellet in the pulp chamber and seal the access hole with a temporary filling material. Recall the patient after a minimum of 48 hours and remove the pellet. Complete the endodontic treatment and restore.
2. Buckley's Formocresol pulpotomy in primary teeth: Isolate the tooth with a rubber dam. Remove visible caries. Remove the coronal pulp with an excavator or bur leaving the radicular pulp. Achieve hemostasis. Moisten small cotton pellets with fomocresol. Blot or wring any excess liquid from the pellet. Place the pellets directly on the radicular pulp stumps for 5 minutes. Place a base or a core in the tooth and restore.

7. STORAGE

Store between 59°-86°F (15°-30°C). Protect from freezing.

Manufactured for:



Sultan Healthcare
1301 Smile Way • York, PA 17404-0807 • USA
Toll Free: 800-637-8582 • Phone: 201-871-1232
Fax: 201-871-0321 • www.sultanhealthcare.com

Made in the USA
0010601DF, R1-080714