

Dry Socket Paste

DIRECTIONS FOR USE

REF 10404 – Dry Socket Paste – 1oz/28g

NPN Number: 02038641

For dental use only.

1. DESCRIPTION

Dry Socket Paste is formulated to relieve the symptoms of alveolar osteitis (“dry socket syndrome”). Dry Socket Paste contains 4% guaicol and 4% eugenol in a petrolatum base.

2. INDICATIONS

Indicated for the relief of symptoms associated with post-extraction alveolar osteitis (“dry socket syndrome”).

3. CONTRAINDICATIONS

This product should not be used with individuals with known sensitivities to guaicol or eugenol. This product should not be used with individuals who are pregnant or suspected to be pregnant without a prior physician consultation.

4. PRECAUTIONS

For professional dental use only. Use only as directed. Not for unsupervised home use.

5. WARNINGS

Do not swallow. Keep out of reach of children. For professional dental use only.

6. DOSAGE AND ADMINISTRATION

1. Establish a diagnosis of alveolar osteitis; typically characterized as a constant dull or throbbing ache, exacerbated by touch, approximately 2 days post extraction. The socket will have no clot present and will usually be packed with food debris.
2. Thoroughly rinse and de-bridge the socket.
3. Introduce Dry Socket Paste into the socket using a flat-bladed instrument or a disposable syringe.
4. Tamp down the Dry Socket Paste, ensuring coverage of all exposed bone.
5. Do not cover the Dry Socket Paste.
6. Dry Socket Paste will remain in the extraction socket 3 to 5 days and will gradually wash out as the socket heals – there is no need for a separate visit to remove the product.

7. STORAGE

Store between 59°-86°F (15°-30°C). Protect from freezing.



Sultan Healthcare
1301 Smile Way • York, PA 17404-0807 • USA
Toll Free: 800-637-8582 • Phone: 201-871-1232
Fax: 201-871-0321 • www.sultanhealthcare.com

Made in the USA
0010404DF, R1-080714