

Eucalyptol, USP

DIRECTIONS FOR USE

REF – 10501 – 1 oz / 30 ml

For dental use only.

Eucalyptol is supplied as a general solvent that can be used for cleaning up certain hardened cements and materials from tools and mixing equipment. It may also be used to soften and dissolve gutta percha during root canal re-treatment procedures. This product is for professional use only. Observe all warnings and precautions on this sheet, the product label and the MSDS sheet.

CAUTION

Do not take internally. Avoid breathing vapor. Avoid skin contact.

FLAMMABLE

Keep away from open flame or heat sources.

WARNING

Observe all warnings and precautions on this sheet, the product label and the MSDS sheet.



Sultan Healthcare
1301 Smile Way • York, PA 17404-0807 • USA
Toll Free: 800-637-8582 • Phone: 201-871-1232
Fax: 201-871-0321 • www.sultanhealthcare.com

Made in the USA
0010501DF, R1-092414