ENGLISH

Topex® Prophylaxis Paste

Unit dose cup: AD30000, AD30001, AD30002, AD30004, AD30007, AD30008, AD30009, AD30011, AD30012, AD30014, AD30015, AD30017, AD30018, AD30019, AD30025, AD30029, AD30041, AD30042
Jar: AD30021, AD30022, AD30024, AD30031, AD30032, AD30034

NPN 00888982

INDICATIONS FOR USE

For cleaning and polishing procedures as part of a professionally administered prophylaxis treatment.

NOTE: Prophylaxis pastes containing fluoride are not considered a replacement for other forms of fluoride therapy.

CONTRAINDICATIONS

This product should not be used in individuals with a known sensitivity to any component of this product.

WARNINGS

Do not intentionally misuse by swallowing. Excessive ingestion of fluoride can cause nausea. Topex® prophylaxis paste should be expectorated and NOT SWALLOWED. For professional dental use only.

PRECAUTIONS

Use only as directed. Not for unsupervised home use. Keep out of reach of children. Do not re-use individual prophylaxis paste cups or disposable prophylaxis angles on multiple patients. If using this product from a jar, dispense a small amount necessary for the procedure and re-seal the jar. Do not dip the prophylaxis cup directly into the jar.

ADVERSE REACTIONS

Patients who have sensitive digestive systems may experience nausea.

DOSAGE AND ADMINISTRATION

BULK JARS: Dispense a small amount necessary for the procedure and re-seal the jar. Do not dip the prophylaxis cup directly into the jar.

UNIT DOSE: Each unit dose cup contains enough prophylaxis paste for (1) prophylaxis procedure. Do not re-use individual prophylaxis paste cups or disposable prophylaxis angles on multiple patients.

DIRECTIONS: Use any commercially available prophylaxis angle, rubber cup combination. To fill the cup, press the cup face into the paste to load. Apply the cup to the tooth surface and use gentle strokes with the handpiece running. Note that speeds above 3000 RPM should be avoided and use caution to avoid overheating the tooth.

Rinse excess material from the teeth and suction any remaining paste or saliva – do not allow the patient to swallow.

COMPOSITION

Medicinal ingredients: 1.23% Fluoride Ion (sodium fluoride)

Non-medicinal ingredients: Glycerin, Pumice, Glycol 1450, Glycol 400, Purified water, Monosodium Phosphate, Veegum D, Agar, Sodium Benzoate, Zinc Stearate

MINT: Sodium saccharin, Mint flavor, FD&C Blue #1, FD&C Yellow #10, Titanium dioxide

CHERRY: Sucralose, Cherry flavor, FD&C Red #40, FD&C Red #22, FD&C Blue #1, Titanium dioxide

PINA COLADA: Sodium saccharin, Pina Colada flavor, FD&C Yellow #6, Titanium dioxide

BUBBLE GUM: Sodium saccharin, Bubble Gum flavor, FD&C Red #22, Titanium dioxide

CHOCOLATE MINT: Sodium saccharin, Chocolate flavor, Mint flavor, FD&C Blue #1, Titanium dioxide

VANILLA: Sodium saccharin (Coarse, Medium)/Sucralose (Fine), Vanilla flavor, Titanium dioxide

STRAWBERRY: Sodium saccharin (Coarse, Medium)/Sucralose (Fine), Strawberry flavor, FD&C Red #40, Titanium dioxide

CHOCOLATE: Sucralose, Chocolate flavor, Lake Red #40, Lake Blue #1, Lake Yellow #6

CINNAMINT™: Sodium saccharin (Coarse)/Sucralose (Medium), Cinnamon flavor, FD&C Red #40, FD&C Yellow #6, Titanium dioxide

REALLY RASPBERRY: Sucralose, Raspberry flavor, FD&C Red #40

ROOT BEER FLOAT™: Sucralose, Root Beer flavor, Lake Red #40, Lake Blue #1, Lake Yellow #6

STORAGE

Store at normal room temperature 20° - 25°C (68° - 77°F).

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